



*Vaginal ovules with
Hyaluronic Acid
and Propolis*

INSTRUCTIONS FOR USE

1. Product description

Apivaginal with Hyaluronic Acid and Propolis is a medical device in the form of ovules. **One ovule contains:** 13 mg of Hyaluronic Acid in the form of Sodium Hyaluronate and 65 mg of Propolis in the form of Ethanolic Propolis Extract. After inserting into the vagina, the ovule dissolves, creating a moisturising and protective gel.

2. Ingredients:

Hyaluronic Acid Sodium Salt, Ethanolic Propolis Extract, Polyoxyethylene Glycols.

3. How it works

After dissolving in the vagina, the ovule forms a sticky hydrogel that coats the mucous membranes.

The Hyaluronic Acid combines with the secretions of the vaginal mucosa to create a gel coating. This layer:

- has a moisturising effect
 - helps the vagina to return to its normal state
 - protects the mucosa against irritation
 - facilitates activation of natural regeneration.
-

4. Uses

This medical device is for:

- the relief of ailments around the labia and vagina, such as dryness, itching, irritation, burning, and redness
 - use after gynecological procedures and surgeries.
-

5. Directions for use

Insert one ovule a day at night for a period of 5 days, until the adverse changes in the vagina or vulva area have disappeared.

In order to remove the ovule from the foil: [as shown in the pictograms]

- remove one plastic shell container from the foil strip
- part the foil leaves and carefully open the plastic shell container
- remove the ovule and insert it into the vagina immediately.

It is best to insert the ovule in the supine position (lying on your back), with thighs slightly raised.



6. Warnings

You should not have sexual intercourse for the duration of the treatment. The course of treatment should be timed so as not to be interrupted by menstruation.

Pregnant and breastfeeding women should consult a doctor before use.

Do not use if allergic to any of the ingredients.

If ailments worsen after use of the product, or eczema, swelling or other previously non-existent symptoms appear in the vulva area, discontinue use and consult a doctor.

If the symptoms improve after using the product, then reappear after 5 days, seek medical help rather than taking another course of the product.

Do not use the product with other preparations administered vaginally.

Keep out of reach of children.

In order to avoid possible staining of linen and to help maintain hygiene, it is recommended that panty liners are worn for the duration of the treatment.

Do not use if any part of the packaging is damaged.

Do not use after the expiry date.

7. Storage

Do not store above 25°C and keep only in original packaging. Keep out of direct sunlight.

8. Packaging

This packet contains 5 ovules.

9. Manufacturer



FARMINA SP. Z O.O.

UL. LIPSKA 44

30-721 KRAKÓW

POLAND

Date instructions for use were updated: 01.2019