



*Gel with
Hyaluronic Acid
and Propolis*

INSTRUCTIONS FOR USE

1. Product description

Apivaginal gel is a preparation in the form of a mucoadhesive gel for use on the external genitalia of women (labia, vaginal vestibule). Apivaginal gel moisturises the mucous membranes, reduces irritation and soothes the symptoms of inflammation.

2. Ingredients

Purified Water, Glycerol, Propylene Glycol, Triethanolamine, Carbomer, Lactic Acid, Propolis Ethanol Extract, Hyaluronic Acid Sodium Salt.

3. How it works

Apivaginal gel creates a protective layer when it comes into contact with the mucosa, moisturising and restoring elasticity. It encourages tissue regeneration processes. Helps return the vaginal mucosa and skin of the labia to its natural physiological state.

4. Uses

Apivaginal gel:

- relieves discomfort around the labia and vestibule, such as itching, burning, swelling, and redness
 - restores the normal physiological condition of the skin around the external genitalia.
-

5. Directions for use

A small amount of gel should be applied to the mucosa of the vaginal vestibule and labia. Depending on the severity of the symptoms, use 1–3 times a day until the disappearance of adverse changes. Do not use for longer than 10 days.

6. Warnings

Do not use if allergic to any of the ingredients.

Pregnant and breastfeeding women should consult a doctor before use.

If symptoms persist or worsen after a 10-day treatment, or symptoms worsen after use of the product, or eczema, swelling or other previously non-existent ailments appear in the vulva area, discontinue use and consult a doctor.

Keep out of reach of children.

In order to avoid possible staining of linen and to help maintain hygiene, it is recommended that panty liners are worn for the duration of the treatment.

Do not use the product with other preparations applied to the external genitalia.

Do not use after the expiry date.

7. Storage

Keep in a dark, cool place. Do not store above 25°C.

8. Packaging

This pack contains 20 g of gel.

9. Manufacturer



FARMINA SP. Z O.O.

UL. LIPSKA 44

30-721 KRAKÓW

POLAND

www.apivaginal.com

Date instructions were updated: 01.2019

MEDICAL DEVICE

